TECH-ATTACH Disposable Connector Block

Contents

en	TECH-ATTACH Disposable Connector Block	1
cs	Jednorázový blok konektorů TECH-ATTACH	4
da	TECH-ATTACH-forbindelsesblok til engangsbrug	7
de	TECH-ATTACH Einweg-Verbindungsblock	10
el	Αναλώσιμη οριολωρίδα TECH-ATTACH	14
es	Bloque conector desechable TECH-ATTACH	18
fr	Bloc de connecteurs jetable TECH-ATTACH	22
it	Blocco connettori monouso TECH-ATTACH	26
nl	Wegwerpbaar TECH-ATTACH-connectorblok	30
pt	Bloco conector descartável TECH-ATTACH	34
tr	TECH-ATTACH Tek Kullanımlık Konektör Bloğu	38

TECH-ATTACH Disposable Connector Block

Symbol Definitions / Definice symbolů / Symboldefinitioner / Symboldefinitionen / Ορισμοί συμβόλων / Definiciones de símbolos / Définitions des symboles / Definizioni dei simboli / Definities van symbolen / Definições dos símbolos / Sembol Tanımları

Manufacturer / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Fabricant / Fabbicante / Fabrikant / Fabricante / Üretici

Use-by date / Datum použitelnosti / Udløbsdato / Haltbarkeitsdatum / Ημερομηνία λήξης / Fecha de caducidad / Date limite d'utilisation / Data di scadenza / Uiterste gebruiksdatum / Data de validade / Son kullanma tarihi

Authorized representative in the European Community / Zplnomocněný zástupce v Evropském společenství / Autoriseret repræsentant i EU / Bevollmächtigter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Mandataire dans la Communauté européenne / Mandatario nella Comunità Europea / Gemachtigde vertegenwoordiger in de Europese Gemeenschap / Mandatário na Comunidade Europeia / Avrupa Topluluğu yetkili temsilcisi

Do not use if package is damaged / Nepoužívejte, pokud je obal poškozený / Må ikke anvendes, hvis pakken er beskadiget / Inhalt bei beschädigter Verpackung nicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / No utilice si el envase está dañado / Ne pas utiliser si le conditionnement est endommagé / Non usare se la confezione è danneggiata / Niet gebruiken als de verpakking is beschadigd / Não utilizar se a embalagem estiver danificada / Ambalaj hasarlıysa kullanmayın

Batch Code / Kód šarže / Batchkode / Chargenbezeichnung / Κωδικός παρτίδας / Código de lote / Code de lot / Codice lotto / Partijcode / Código de lote / Parti Kodu

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DESCRIPTION

The TECH-ATTACH Disposable Connector Block is one component of the TECH-ATTACH Connection System. The system is designed to provide an easy method for making a secure connection between an Ad-Tech® electrode and standard clinic EEG monitoring and/or stimulation equipment.

The TECH-ATTACH Disposable Connector Block is a clear block with an inlet to insert the tail of an Ad-Tech® electrode. It is designed to be used with TECH-ATTACH Cables.

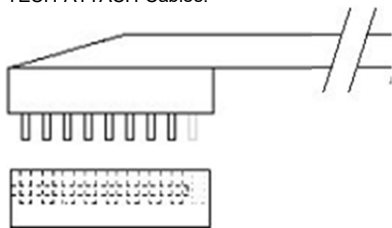
Figure 1 - TECH-ATTACH Disposable Connector Block (lower) shown with the terminal block of the cables.

Design Variants

The most common variants involve the number of contacts. At present, Ad-Tech® electrodes have a maximum number of 16 contacts per tail and often the blocks are built to match. Blocks can be slotted or drilled through to accommodate macro-micro depth electrodes or packaged as individual units. The TECH-ATTACH Disposable Connector Block are also available with a smaller diameter hole in the block and with the pins more closely spaced to accept smaller sized Foramen Ovale tails. These variants do not alter the indications or the intended use of the TECH-ATTACH Disposable Connector Block.

Depth Electrodes	Spencer Probe
	Macro-Micro
	Foramen Ovale
Subdural Electrodes	Grid
	Strip
	Dual-Sided Interhemispheric
	Multi-Strip and Split Grid

Table 1 - Compatible Ad-Tech® Electrodes

The TECH-ATTACH Disposable Connector Blocks are supplied sterile. There are two (2) blocks per pouch and five (5) pouches per box.

INDICATIONS AND INTENDED USE

The TECH-ATTACH Disposable Connector Block is designed to be used with TECH-ATTACH Cables to provide a safe, secure, and easy method to connect Ad-Tech® electrodes with typical EEG monitoring and/or stimulation equipment.

INTENDED USER

This product should only be used by a healthcare professional trained in the use of the TECH-ATTACH Connection System.

PATIENT TARGET GROUP

This device is used in patients diagnosed with intractable epilepsy who are undergoing brain mapping procedures to define the location of epileptogenic foci.

EXPECTED CLINICAL BENEFIT

The TECH-ATTACH Disposable Connector Block provides a secure connection between the Ad-Tech® electrode and standard EEG recording equipment.

CONTRAINDICATIONS

None known.

WARNINGS

- TECH-ATTACH Disposable Connector Blocks are provided STERILE. Do not use if packaging is open or damaged.
- Single use only. Do not attempt to resterilize or reuse. Reuse may cause contamination risk to the patient and the block may malfunction generating unreliable EEG results.
- Disconnect from monitoring equipment during cardiac defibrillation.

CAUTIONS

- Federal law (USA) restricts this device to sale by or on the order of a physician.
- A direct pull or stress on the electrode may cause damage to the electrode or a loss of contact recordings. If the patient will become mobile, place and secure a strain relief using the EEG cable.

ADVERSE EVENTS

None known.

INSTRUCTIONS FOR USE


To connect electrode to cable:

1. Insert the electrode tail into the inlet of the connector block as shown. There is only one inlet and the electrode tail should easily fit.


Behnke Fried (BF) Electrodes: When placing the block on a BF electrode tail, ensure the single pin hole on the block is located opposite the recording end (see


circled pin hole location in the figure below). When properly placed, the electrode tail enter and the cable will exit the connection block from opposite sides. Incorrect block placement will result in contact misidentification against the Code Chart provided with the BF electrode.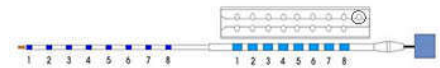


2. Confirm all contact collars on the electrode tail are aligned with the dual pin holes in the connector block. The contact collars may be viewed from the side or top of the connector block.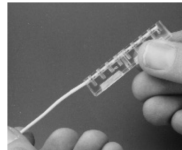

3. Keeping the contact collars aligned with the dual pin holes, carefully place the terminal block of the cable into the connector block.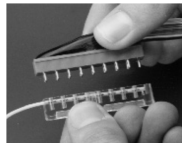

4. Firmly press the terminal block on the cable into the connector block. The connection of the electrode to the cable is complete. When properly connected, the electrode tail stays in place. If the connection is incorrect or reversed, one gold pin from the terminal block of the cable will be outside the connector block and the blocks will be misaligned.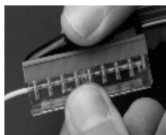

Important Note: For chronic use, it is important to place and secure a strain relief using the EEG cable. A direct pull on the electrode may damage the electrode or cause loss of contact recordings.


To disconnect electrode from cable:

NOTE: Take caution not to pull or place stress directly on the electrode tail. A direct pull or stress may cause damage to the electrode.


1. Grasp the distal section of the connector block. Pull the block connector and the terminal block approximately 1mm - 2mm apart.

2. Grasp the proximal section of the connector block. Pull the block connector and the terminal block approximately 1mm - 2mm apart to create a parallel gap between them. If you hear a click, it is the gold pins in the terminal block releasing the contact collars of the electrode tail.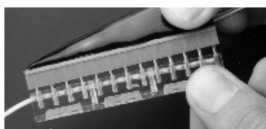

3. Grasp the center section and pull apart the block connector and the terminal block. The terminal block and the block connector should easily pull apart. If the blocks do not release easily, repeat steps 1 and 2 until they release.

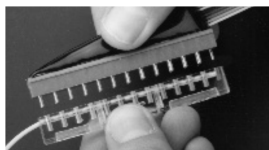STORAGE

Store and use TECH-ATTACH Disposable Connector Blocks in a dry, clean location maintained at typical hospital/office ambient temperature and humidity conditions.

DEVICE DISPOSAL

This device does not require any special disposal procedures, however, if the device has been contaminated with potentially infectious substances of human origin (such as bodily fluids), it should be handled according to hospital protocol for infectious medical waste

INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

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© 2021 AD-TECH
AD-TECH® Medical Instrument Corporation
E C REP Limited